Hyperinflation System with Adjustable Pressure Limiter (APL)

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INDICATIONS FOR USE:

The XXX NuFLO2 Hyperinflation Bag System is a pulmonary assist device intended to provide controlled or assisted ventilation to patients.

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1.

This product must only be used by medical personnel well trained in techniques of positive pressure ventilation with a hyperinflation bag system.

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2.

Never wait to begin positive pressure ventilation if a hyperinflation system is not immediately available or cannot be used effectively (see your department manual for accepted positive pressure ventilation procedures).

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3.

While using supplemental oxygen, do not allow smoking or use unit near sparking equipment, open flame, oil or other flammable chemicals.

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4.

Should not be used in toxic or hazardous atmospheres.

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5.

Verify proper function of the hyperinflation bag system and verify clear patient airway by monitoring for the following:

- That the patient is being ventilated, as indicated by rise and fall of chest.
- That the dark blue (high pressure) or light blue (low pressure) valve rotates and the mask and patient valve are free from obstruction.
- To clear valve obstructions, shake briskly.
- Product should be constantly monitored while in use.
- Gas source must be attached and flowing for hyperinflation bag system to operate properly.
- That the breathing bag is securely and properly attached to the valve body.

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6.

This product should be used in conjunction with a manometer pressure gauge to determine appropriate delivery pressure.

Pressure may be monitored with Mercury Medical Single-Patient-Use Manometer Pressure Gauge that may be incorporated on the APL valve.

Units that do not feature the built-in manometer MUST be monitored with a line attached from the pressure port to a manometer pressure gauge.

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7.

When using the NuFLO2 hyperinflation system, it is recommended that clinicians do not exceed 10 liters of flow per minute or 40 cm of H₂O pressure unless the user determines that higher pressures are required to effectively ventilate the patient based on clinical experience and judgment.

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8.

NuFLO2 hyperinflation units with dark blue APL knob can be set to a maximum of approximately 60 cm of H₂O while the units with the light blue APL knob can be set to a maximum of approximately 35 cm of H₂O.

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9.

Reuse of this device may pose a risk of cross-contamination and the device may not perform as intended.

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10.

Disposal of this device should be handled in accordance with hospital protocol.

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11.

This device is assembled with PVC components which include low levels of a phthalate as indicated.

Consideration should be used when treating children or pregnant or nursing women.

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